


ELMED

ELMED SILASTIC RING APPLICATOR FOR FALLOPIAN TUBULAR STERILIZATION

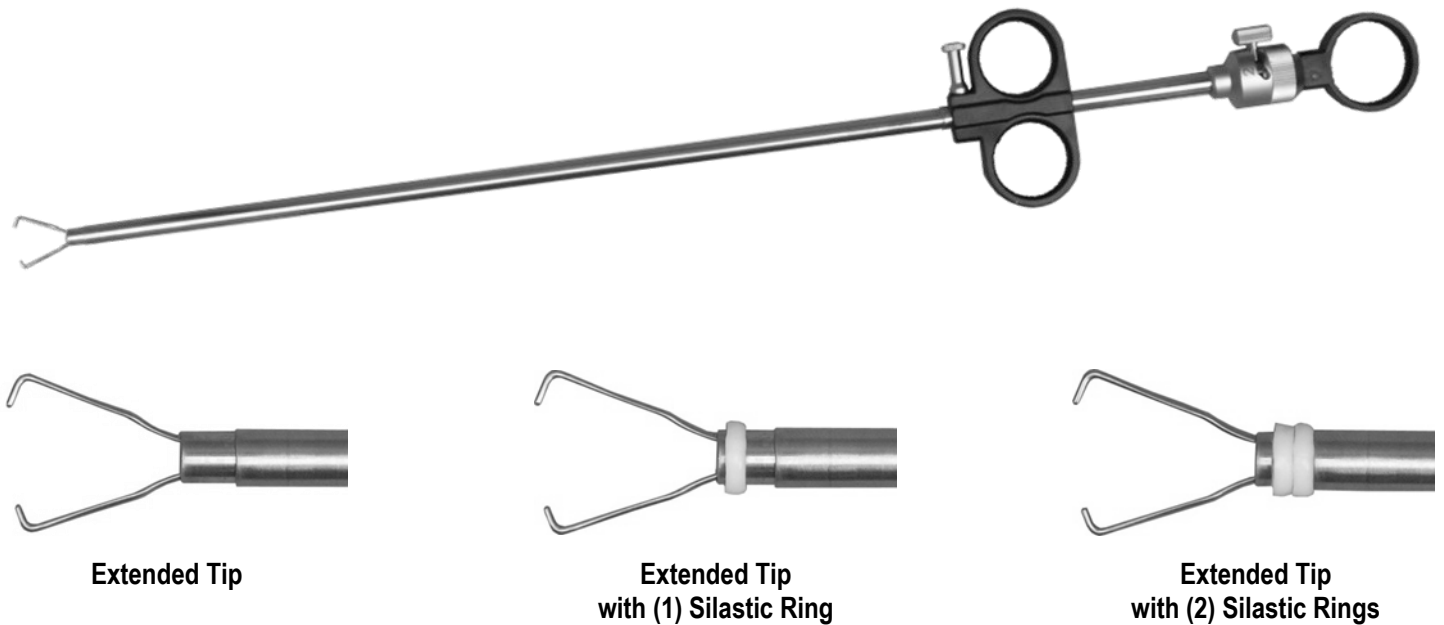

Female sterilization by means of **Silastic Ring Applicator** is clinically well-established based on more than 35 years of experience. Our instruments are completely reusable and fully autoclavable. They are manufactured of the highest grade of materials by experienced and skilled craftsmen.



The **Elmed Silastic Ring Applicators** are made for single or dual ring application with one single insertion. The instruments are available in three different lengths for use through the operating scope (single puncture approach) or through trocar sleeve in conjunction with a laparoscope (multi-puncture approach).



ALSO AVAILABLE IN THE ACMI™ STYLE: 8MM, 27CM...LIST NO. 52208-501

LIST NO.	DESCRIPTION
52105-500	6 MM Ø, 40 CM SILASTIC RING APPLICATOR FOR SINGLE & DUAL RING APPLICATION; USE W/ OPERATING LAPAROSCOPE WITH 6MM INSTRUMENT CHANNEL
52205-501	6 MM Ø, 33 CM SILASTIC RING APPLICATOR FOR SINGLE AND DUAL RING APPLICATION; USE WITH LAPAROSCOPE AND 6MM Ø TROCAR SLEEVE (SEE ELMED DATA SHEET FOR APPROPRIATE SLEEVE).
52208-500	8 MM Ø, 33 CM SILASTIC RING APPLICATOR FOR SINGLE AND DUAL RING APPLICATION; USE WITH LAPAROSCOPE AND 8MM Ø TROCAR SLEEVE (SEE ELMED DATA SHEET FOR APPROPRIATE SLEEVE). SAME DIAMETER AS ACMI.
52208-501	8 MM Ø, 27 CM SILASTIC RING APPLICATOR FOR SINGLE & DUAL RING APPLICATION; USE W/ LAPAROSCOPE & 8MM TROCAR SLEEVE W/ TRIANGLE HANDLE (SEE ELMED DATA SHEET FOR APPROPRIATE SLEEVE). SAME DIAMETER AS ACMI.
52005-500	SILASTIC RING (package of 10) FOR TUBAL LIGATION Non-sterile, autoclavable
52005-501	SILASTIC RING APPLIER AND GUIDE CONE SET (Note: (1) set comes with new instrument)



EFFECTIVE STERILIZATION PROCEDURE WITH SILASTIC RING APPLICATOR

A ten year study has proven the **Silastic Ring Applicator** to be one of the most reliable methods of mechanical tubal occlusion. Its high rate of success has contributed to this technique's worldwide acceptance.

ELMED offers the greatest selection of instrumentation. This selection enables your physicians to choose from a variety of instrumentation including single or double puncture application.

This simple mechanical technique is the natural choice for safe, uncomplicated female sterilization. The Silastic ring is an inert, radio opaque silicone band that is simply applied to a looped section of the fallopian tube with the **Silastic Ring Applicator**.


The procedure provides high potential for reversibility. The success of surgical anastomosis is most dependent upon the skill of the surgeon.

ILLUSTRATION OF THE TUBAL LIGATION PROCEDURE:

Single puncture using 12 mm operating laparoscope.

Double puncture requiring a 10 mm laparoscope and 7 mm Ø trocar sleeve.

The **Silastic Ring Applicator** contains forceps to grasp the fallopian tube 3 to 4 cm from the cornual area. The fallopian tube is gently drawn into the inner cylinder of the instrument forming a knuckle. The **Silastic Ring** located on the inner cylinder is released onto the knuckle, occluding the base. This procedure is repeated on the second Fallopian tube. Once in place, the **Silastic Ring** will not slip.

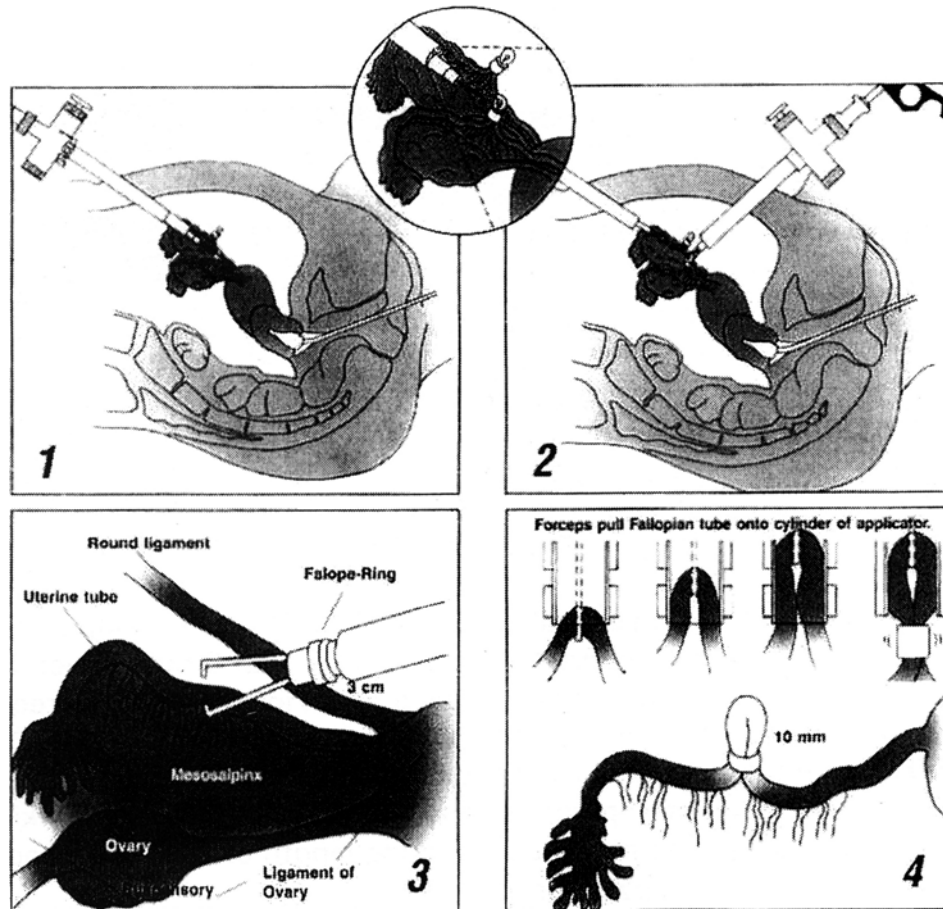

WE DESIGN, MANUFACTURE & SELL THE TOOLS THE SURGEONS USE

Elmed Incorporated
35 N Brandon Drive
Glendale Heights, IL 60139
USA

We subscribe to Cost
Containment and Protection
of the Environment.



Therefore, we manufacture
reusable products for a cleaner
world.

Phone - (224) 353-6446
Fax - (224) 653-8178
Email - medical@elmed.com
Web - www.elmed.com