

OPERATING ROOM STOOLS

FOR SURGEONS & ANESTHESIOLOGISTS

INSTRUCTIONS FOR USE

OUR ELMED STOOLS ARE ERGONOMICALLY DESIGNED AND CONSTRUCTED FOR MAXIMUM DURABILITY IN THE OPERATING ROOM SETTING. IT PROVIDES MAXIMUM COMFORT AND PRECISE STABILITY FOR THE SURGEON OR ANESTHESIOLOGIST DURING SURGICAL PROCEDURES THAT REQUIRE LONG EXTENDED SITTING TIMES. THE ELMED SURGEONS STOOLS ARE A QUALITY PRODUCT INCORPORATING THE FINEST WORKMANSHIP, UTILIZING ONLY THE HIGHEST QUALITY MATERIAL. PLUS, HANDS FREE OPERATION IN STERILE FIELD.

THE MOST IMPORTANT FEATURE: HANDS FREE OPERATION IN A STERILE FIELD

THE SURGEON'S HANDS ARE ALWAYS STERILE. HE NEVER HAS TO USE THEM, REQUEST OUTSIDE ASSISTANCE OR REMOVE HIM/HERSELF FROM THE STOOL TO RAISE IT UP OR DOWN.

THE ELMED STOOL FEATURES A HEEL/TOE CONTROLLED HEIGHT ADJUSTMENT SYSTEM.

TO OPERATE: SLIGHTLY SHIFT YOUR WEIGHT TO THE LEG RESTING ON THE FLOOR. WITH THE HEEL OR TOE OF THE OTHER FOOT, DEPRESS THE STEEL RING AT THE BASE. THE SEAT SECTION WILL AUTOMATICALLY RISE OR LOWER TO THE DESIRED HEIGHT.

FEATURES:

- SOLID CIRCULAR HEAVY STEEL POWDER COATED BASE (20" DIA.) WITH EMBEDDED BLACK RUBBER BUMPER WHICH ALSO SERVES AS A CONVENIENT FOOTREST WHILE ADDING RIGIDITY TO THE BASE.
- FIVE HEAVY-DUTY, ELECTRICALLY CONDUCTIVE BALL BEARING, SELF-LOCKING CASTERS GIVE GREAT MOBILITY WITH ABSOLUTE TILT-FREE STABILITY THAT HAVE A BUILT-IN SPRING MECHANISM LOCKING THE WHEEL WHEN NO WEIGHT IS ON THE STOOL.
- HEIGHT ADJUSTMENT BY FOOT-ACTIVATED AIR CYLINDER BETWEEN 550-780MM (21.5" - 32")
- THE SEAT SECTION, ROUND 14" DIA. OR SADDLE SHAPED 16"x15" IS COVERED WITH AN ELECTRICALLY CONDUCTIVE HEAVY-DUTY, EASY TO CLEAN NAUGAHYDE OVER A HEAVY FOAM PAD PROVIDING OPTIMUM COMFORT AND DURABILITY.
- THE BACK SUPPORT (MODELS 222-167 & 222-168) ARE ADJUSTABLE IN HEIGHT AND ARE MADE OF THE SAME FOAM PADDING AND NAUGAHYDE COVER AS THE SEAT SECTION. IT PROVIDES EXCELLENT BACK SUPPORT AND MAY ALSO BE USED AS AN ARM SUPPORT IN CERTAIN POSITIONS,
- THE STOOL IS COMPLETELY ELECTRICALLY CONDUCTIVE BOTH SEAT SECTION AND CASTERS MEETING ALL THE SAFETY REQUIREMENTS.
- WEIGHT LIMIT IS 300 LBS.



ITEM NO. **222 165**
OUR MOST POPULAR MODEL

CLEANING:

- FOR UPHOLSTERY, WE RECOMMEND USING A WEAK ALKALINE DETERGENT, E.G. MILD CLEANING AGENT OR SOAP AND WATER MAY ALSO BE USED.
- FOR THE STAINLESS STEEL, WE RECOMMEND USING ANY COMMERCIALLY AVAILABLE STAINLESS STEEL CLEANER.

WE DESIGN, MANUFACTURE, & SELL THE TOOLS THE SURGEONS USE

ELMED INC.
35 N. BRANDON DR.
GLENDALE HEIGHTS, IL
60139
USA

WE SUBSCRIBE TO COST
CONTAINMENT AND
PROTECTION OF THE
ENVIRONMENT



THEREFORE, WE
MANUFACTURE
REUSABLE PRODUCTS
FOR A CLEANER
WORLD

PH (224) 353-6446
FAX (224) 653-8178
EMAIL
MEDICAL@ELMED.COM
WWW.ELMED.COM