



COHEN CANNULA WITH FIXED ACORN CLEANING & DISASSEMBLY

THIS INSTRUMENT CAN BE STEAM AUTOCLAVED AT 134 C (272 F), COLD SOAKED OR GAS STERILIZED.


THE COHEN CANNULA HAS ONE NON-REMOVABLE STAINLESS STEEL ACORN WITH MALLEABLE STAINLESS STEEL TIPS FOR THE NULLIPAROUS AND PAROUS CERVIX. THE STEEL TIP HAS SIDE OPENINGS, IS CLOSED AT THE END AND IS TAPERED TO PREVENT TRAUMA TO THE ENDOMETRIAL CAVITY.

THE INSTRUMENT IS LEAK-PROOF, FITTING SNUGLY AGAINST THE CERVIX AND HELD IN POSITION BY MEANS OF A CONCEALED TENSION SPRING. A SIMPLE TRIGGER MECHANISM IN THE HANDLE ENGAGES THE LOCK OF A STANDARD 9½" (24 CM) SINGLE TENACULUM FORCEPS.

THERE IS A FEMALE LUER TIP AT THE END OF THE CANNULA THAT ADAPTS TO A LUER SYRINGE OR DETACHABLE STOPCOCK WITH LUER TIP.

DISASSEMBLY:

1. LOOSEN SPRING HOUSING (2) BY HOLDING CANNULA AT FINGER GUIDE (3) SECURELY AND TURNING SPRING HOUSING COUNTERCLOCKWISE (5).
2. WHILE SPRING HOUSING (5) IS ROUGHLY HALFWAY UNSCREWED, PULL FORCEFULLY FROM FINGER GUIDE (3) AND FIXED ACORN (1). (IT WILL CLICK APART IF PULLED CORRECTLY.)
3. PROCEED BY COMPLETELY UNSCREWING SPRING HOUSING (5).

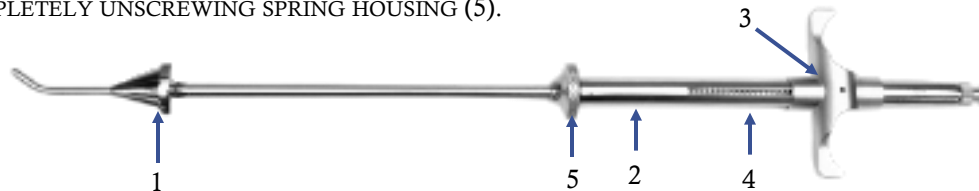

- ONCE YOU HAVE DISASSEMBLED THE INSTRUMENT, RINSE WITH DEIONIZED WATER TO REMOVE ANY GROSS DEBRIS.
- HAND WASH ALL PARTS USING NEUTRAL PH DETERGENT OR ENZYMATIC CLEANER AND SOFT BRUSH, BEING SURE TO FLUSH ALL LUMENS AND CHANNELS WITH THE SOLUTION.
- THOROUGHLY CLEAN INSIDE OF OUTER TUBE AND CANNULA BY USING A SOFT BRUSH AND FLUSHING SOLUTION THROUGH IT.
- AVOID THE USE OF STEEL WOOL, WIRE BRUSHES AND/OR ABRASIVE DETERGENTS.
- AFTER RINSING, INSPECT ALL COMPONENTS FOR BROKEN OR WORN PARTS THAT MAY INHIBIT THE FUNCTION.

ASSEMBLY:

1. TENSION SPRING (4) MUST BE PLACED ON INNER ROD OF CANNULA.
2. HOLD ACORN (1) AT DESIRED POSITION WHILE SECURING AND TURNING SPRING HOUSING (5).

ORDERING INFORMATION

52001-12S	COHEN CANNULA WITH FIXED SMALL STAINLESS STEEL ACORN
52001-12L	COHEN CANNULA WITH FIXED SMALL STAINLESS STEEL ACORN
52003-29	3MM Ã~ CLEANING BRUSH FOR FRONT TUBE OF COHEN CANNULA (QTY: 12/PACK)
52003-31	5MM Ã~ CLEANING BRUSH FOR REAR TUBE OF COHEN CANNULA (QTY: 12/PACK)

WE DESIGN, MANUFACTURE, & SELL THE TOOLS THE SURGEONS USE

ELMED INC.
35 N. BRANDON DR.
GLENDALE HEIGHTS,
IL 60139
USA

WE SUBSCRIBE TO
COST CONTAINMENT
AND PROTECTION OF
THE ENVIRONMENT



THEREFORE, WE
MANUFACTURE
REUSABLE PRODUCTS
FOR A CLEANER
WORLD

PH (224) 353-6446
FAX (224) 653-8178
EMAIL
MEDICAL@ELMED.COM
WWW.ELMED.COM