



## USE & HANDLING INSTRUCTIONS

1. CLEANSE PRODUCT IN A SUITABLE SURGICAL CLEANSER WITH TIP FULLY EXTENDED. CHARRED TISSUE ON THE INSTRUMENT TIP CAN BE REMOVED BY USING AN ABRASIVE PAD OR STIFF BRUSH AND SCOURING POWDER.
2. RINSE THOROUGHLY. REPLACE CLEAR PLASTIC TUBE OVER TIP AND SHAFT (MAKE SURE TIP IS FULLY EXTENDED).
3. MAXIMUM PRODUCT LIFE IS ACHIEVED BY USING ETHYLENE OXIDE GAS (EtO) STERILIZATION. FOLLOW STERILIZER MANUFACTURER'S RECOMMENDED PARAMETERS. HOWEVER, DO NOT EXCEED TEMPERATURES OF 180° F.
4. COLD SOAKING IN STERILIZING SOLUTIONS IS AN ACCEPTABLE ALTERNATE METHOD. BECAUSE SOME STERILIZING SOLUTIONS CONTAIN PENETRANTS, DOUBLE RINSING WITH AGITATION IS RECOMMENDED.
5. REGARDLESS OF STERILIZING METHOD EMPLOYED, PRODUCT SHOULD BE THOROUGHLY DRIED PRIOR TO USE. PARTICULAR ATTENTION SHOULD BE PAID TO THE POWER CORD CONNECTOR.
6. ALWAYS STORE PRODUCT WITH DISTAL ELECTRODE TIP FULLY EXTENDED AND CLEAR PLASTIC TUBE PLACED OVER TIP AND SHAFT.
7. ALWAYS RETRACT ELECTRODE TIP WHEN INSERTING AND REMOVING ELECTRODE FROM THE SCOPE.

**WARNING:** *THESE ELECTRODES ARE DESIGNED FOR USE IN RIGID SCOPES ONLY.*

8. THE RECOMMENDED ACTIVATION CORD FOR THESE PRODUCTS IS ITEM #5260 (ELMED & ELLMAN GENERATORS) OR #5261 (UNITS WITH 8MM DIAMETER BOVIE SOCKET, E.G. VALLEYLAB). ATTACH CORD TO ELECTRODE BY PUSHING THE SMALL TAPERED CORD PIN INTO THE HOLE OF THE METAL CONNECTOR INSIDE THE PROXIMAL END OF THE WHITE HANDLE ASSEMBLY.
9. ***TIP MUST BE FULLY SUBMERSED IN IRRIGATING FLUID WHEN ELECTRICAL CURRENT IS EMPLOYED.*** DO NOT EXCEED ELECTROSURGICAL GENERATOR SETTINGS OF 80 WATTS (COAGULATION). PRODUCT CATALOG NO'S. UET107B, RET107B, RET107F, AND RET107G ONLY MAY ALSO BE OPERATED IN THE CUTTING MODE – DO NOT EXCEED GENERATOR SETTINGS OF 200 WATTS.

**IMPORTANT:** THE HANDLE OF THE UET ELECTRODES MUST BE SUPPORTED AT ALL TIMES. THEY CANNOT HANG BY THEMSELVES, OR THE WIRE WILL BREAK.

**WE DESIGN, MANUFACTURE, & SELL THE TOOLS THE SURGEONS USE**

ELMED INC.  
35 N. BRANDON DR.  
GLENDALE HEIGHTS, IL  
60139  
USA

WE SUBSCRIBE TO COST  
CONTAINMENT AND  
PROTECTION OF THE  
ENVIRONMENT



MADE IN THE USA

THEREFORE, WE  
MANUFACTURE  
REUSABLE PRODUCTS  
FOR A CLEANER  
WORLD

PH (224) 353-6446  
FAX (224) 653-8178  
EMAIL  
MEDICAL@ELMED.COM  
WWW.ELMED.COM